BoneHearing F2

Bone Conduction Sound Amplifier Headphones

User Manual

• WHAT'S IN THE BOX





X1



Charging Cable



Silicone Strap


X1



1. Place the headphones over your ears, ensuring

the ear hooks fit snugly without blocking the ear canal. 2. Adjust the bone conduction units to rest on your cheekbones for optimal sound.

BUTTON CONTROL

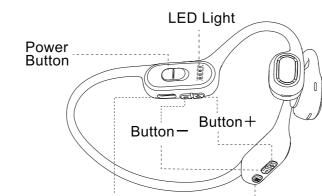


Type-C Charging Port Multi-functional

Button (M)

• SILICONE STRAP USAGE Loosen Tighten

For a smaller head circumference, use this silicone strap accessory to secure headphones better. Tie each end of the strap to the headphone band behind the ears.

• LIGHT SENSOR HEADPHONES

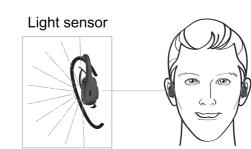

The headset is built-in light sensor to reach the max


- After power-on, the headset will start to work when you wear on the headset.
- When you take off the headset, it will pause operation.

POWER ON/OFF






• The 1st LED flashes in white for 3 times. Prompt: power on pairing.

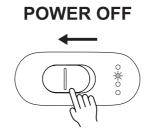


• The 2nd LED flashes in white for 3 times. Prompt: Power off.



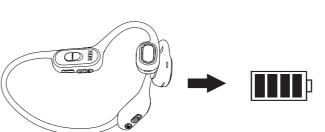

Automatically announce the real-time battery level of the headphones after power on. Battery 100%-70%: battery high. Battery 69%-40%: battery medium. Battery 39%-10%: battery low.

• THREE AMPLIFIER MODES

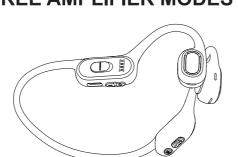

The headphones is built-in 3 different sound modes for amplifying.

Click button M to switch indoor mode / outdoor mode



Press the +/- button on the left/right side to adjust the volume. When the maximum volume is reached, Prompt: Maximum volume.


the left and right ears.

F2 supports independent volume adjustment for

• WIRELESS MODE


- Switch Amplifier / BT Mode Double click the button M. Prompt: Wireless mode / Amplifier mode.
- BT Connection

Select "BoneHearing F2" in BT menu to connect. Prompt: connected.

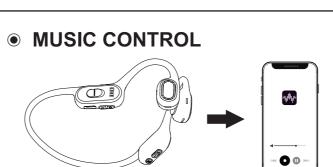

- Volume Up: Click the button + .
- ullet Volume Down: Click the button . Previous Track: Long-press the "+" button on either side.
 Next Track: Long-press the "-" button on either side.
- Play/Pause Music: Click the button M once.
- Answer/Hang Up A Call: Click the button M once. • Reject A Call: Double click the button M.
- Activate Voice Command: Quickly triple-click the button M.

• LOW BATTERY

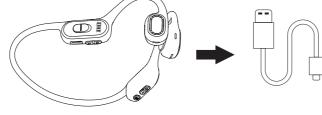

Low Battery • When battery is lower than 20%, prompt:

battery low.

Charging • Please use DC 5V adapter for charging. Green LED is on charging. White LED is on



• CHARGING LED INDICATORS

○ ○ ○ ○ Battery 75-100%

○ ○ ○ ○ ○ Battery 50-75%

 ○ ○ ○ ○ Battery 25-50% ○ ○ ○ ○ ○ Battery 15-25%

after fully charged.

SPECIFICATIONS	
Model	BoneHearing F2
Туре	Bone conduction sound amplifier headphones
BT Version	V5.3
Connection	≈15m
Continuous wireless usage time	≥6h(80%volume)
Continuous amplifying usage time	≥6h(80%volume)
Charging Time	≈1h
Charging Voltage	DC 5V
Waterproof Rating	IPX5

• ATTENTIONS 1. High-volume use can damage your hearing. Follow

- safe volume and time limits and get regular hearing check-ups. 2. Headphones affect your awareness of surroundings,
- please use them with caution. 3. Do not disassemble or modify the headphones.
- 4. Avoid throwing, sitting on, or burning the headphones. 5. Keep headphones away from fire and high-voltage
- equipment. 6. Not for children under ten due to small parts.
- regular waste. 8. Consult a doctor if you have serious hearing issues before using bone conduction headphones.

7. Recycle headphones at an electronics facility, not as

Note: The company is not liable for damage resulting from non-compliance with these warnings.