

①

BoneHearing F2C

Bone Conduction
Sound Amplifier Headphones

User Manual

WHAT'S IN THE BOX

BoneHearing F2C X1 Manual X1 Charging Cable X1

Earplugs X1 Silicone Strap X1

②

HOW TO WEAR

- Place the headphones over your ears, ensuring the ear hooks fit snugly without blocking the ear canal.
- Adjust the bone conduction units to rest on your cheekbones for optimal sound.

BUTTON CONTROL

③

SILICONE STRAP USAGE

For a smaller head circumference, use this silicone strap accessory to secure headphones better. Tie each end of the strap to the headphone band behind the ears.

POWER ON/OFF

POWER ON

- The 1st LED flashes in white for 3 times. Prompt: power on pairing.

POWER OFF

- The 2nd LED flashes in white for 3 times. Prompt: Power off.

④

BATTERY LEVEL NOTIFICATION

Automatically announce the real-time battery level of the headphones after power on.

- Battery 100%-70%: battery high.
- Battery 69%-40%: battery medium.
- Battery 39%-10%: battery low.

THREE AMPLIFIER MODES

The headphones is built-in 3 different sound modes for amplifying.

Click button M to switch indoor mode / outdoor mode/ vocal mode.

⑤

VOLUME CONTROL

F2C supports independent volume adjustment for the left and right ears.

Click the +/- button on the left/right side to adjust the volume. When the maximum volume is reached, Prompt: Maximum volume.

WIRELESS MODE

- Switch Amplifier / BT Mode**
Press and hold the M key. Prompt: Pairing/Wireless mode/Amplifier mode.
- BT Connection**
Select "BoneHearing F2C" in BT menu to connect. Prompt: connected.
- BT & AMP. SIMULTANEOUS**
The headphones default to hearing aids mode when powered on. However, in Bluetooth mode, if the music is paused while listening, it will automatically switch back to hearing aids mode.

⑥

MUSIC CONTROL

- Volume Up: Click the button +.
- Volume Down: Click the button -.
- Previous Track: Long-press the "+" button on either side.
- Next Track: Long-press the "-" button on either side.
- Play/Pause Music: Click the button M once.

CALL CONTROL

- Answer/Hang Up A Call: Click the button M once.
- Reject A Call: Double click the button M.
- Activate Voice Command: Quickly triple-click the button M.

⑦

LOW BATTERY

Low Battery

- When battery is lower than 20%, prompt: low battery.

Charging

- Please use DC 5V adapter for charging. Green LED is on charging. White LED is on after fully charged.

CHARGING LED INDICATORS

- Battery 75-100%
- Battery 50-75%
- Battery 25-50%
- Battery 15-25%
- ✧○○○ LED blinks, please charge.

⑧

SPECIFICATIONS

Model	BoneHearing F2C
Type	Bone conduction sound amplifier headphones
BT Version	V5.3
Connection	≈15m
Continuous amplifying usage time	≥8h(80%volume)
Continuous wireless usage time	≥7.5h(80%volume)
Charging Time	50min
Charging Voltage	DC 5V
Waterproof Rating	IPX5
Avg Gain@Vocal Mode	30±3dB

ATTENTIONS

- High-volume use can damage your hearing. Follow safe volume and time limits and get regular hearing check-ups.
- Headphones affect your awareness of surroundings, please use them with caution.
- Do not disassemble or modify the headphones.
- Avoid throwing, sitting on, or burning the headphones.
- Keep headphones away from fire and high-voltage equipment.
- Not for children under ten due to small parts.
- Recycle headphones at an electronics facility, not as regular waste.
- Consult a doctor if you have serious hearing issues before using bone conduction headphones.

Note: The company is not liable for damage resulting from non-compliance with these warnings.